

STATSEAL®

ADVANCED TOPICAL DISC PRODUCTS

StatSeal® is intended for use under the care of a healthcare professional for the temporary external control of bleeding from sheath and catheter removals. Each application includes one (1) **StatSeal Advanced Disc** and one (1) product description label.

INSTRUCTIONS FOR USE

1. Prepare skin around puncture site per facility protocol.
2. Pull sheath back 2-3 cm.
3. Open sterile packaging and remove contents. Remove paper backing from disc to expose adhesive.
4. Place disc with label side up over sheath at puncture site. If using StatSeal Advanced RAD Disc, center disc over puncture site, lengthwise along the artery.
5. Slowly pull sheath while applying **semi-occlusive** pressure directly over arteriotomy and puncture site. Allow a small amount of blood to come in contact with disc in order to form a seal.
6. Maintain **continuous, semi-occlusive** pressure over disc and arteriotomy site for ≥ 5 minutes. Extend pressure times when using a mechanical pressure device or as clinically necessary.
7. Release pressure slowly to check for hemostasis. Apply additional **semi-occlusive** pressure as necessary.
8. Place transparent dressing per facility protocol. Apply product description label over dressing.
9. Leave disc in place until scheduled dressing change.

STATSEAL®

INGREDIENTS: Hydrophilic polymer and potassium ferrate

WARNINGS: FOR EXTERNAL USE ONLY. Not intended to replace stitches. Do not apply to visibly infected wounds. May be harmful if inhaled or ingested. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Depending on patient condition and amount of pressure, an adverse event including artery occlusion, hypodermic hematoma, hemorrhage, pain or numbness may occur. Check hemostasis progress and adjust pressure as clinically necessary.

CAUTION: Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged or opened. Discard contents after use. After opening, contents are no longer sterile.

 Biolife, LLC

8163 25th Court East, Sarasota, FL 34243, USA
+1 941.360.1300 • customer.care@biolife.com

**The symbols glossary is provided electronically at:
www.statseal.com/symbols**



2460



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands