

# STATSEAL®

StatSeal® is intended for use under the care of a healthcare professional for the temporary external control of bleeding from sheath and catheter removals.

Each application includes one (1) **StatSeal Disc** and one (1) product description label.

## Instructions for Use

1. Pull sheath back 2-3 cm.
2. Open sterile packaging and remove disc. Remove paper backing from disc.
3. Place disc with label side up over sheath at access site.
4. Pull sheath slowly while applying pressure directly over the arteriotomy and puncture site. Allow a small amount of blood to come in contact with disc in order to form a seal.
5. Hold **continuous**, semi-occlusive pressure over disc and arteriotomy site for  $\geq 5$  minutes. Extend hold times as necessary for larger sheaths or anticoagulated patients.
6. Release pressure slowly to check for hemostasis. Apply additional pressure as necessary.
7. Place transparent dressing per protocol. Apply product description label over dressing.
8. Leave disc in place for 24 hours.

Additional instructions and procedural tips are available at:  
[www.statseal.com](http://www.statseal.com)

INS-SS-01.2

# STATSEAL<sup>®</sup>

**Ingredients:** Hydrophilic polymer and potassium ferrate

**WARNINGS: FOR EXTERNAL USE ONLY.** Not intended to replace stitches. Do not apply to visibly infected wounds. May be harmful if inhaled or ingested. Avoid aspirating product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Keep out of reach of children.

**CAUTION:** Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged or opened. Discard contents after use. After opening, contents are no longer sterile.

**The symbols glossary is provided electronically at:  
[www.biolife.com/symbols](http://www.biolife.com/symbols)**



Biolife, LLC

8163 25th Court East, Sarasota, FL 34243, USA

+1 941.360.1300 • [customer.care@biolife.com](mailto:customer.care@biolife.com)



0543



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands

Patents 6,521,265, 6,187,347, 8,007,833 and 8,961,479

Other U.S. and international patents pending.