

STATSEAL®

TOPICAL DISC PRODUCT FAMILY

StatSeal® is intended for use under the care of a healthcare professional for the temporary external control of bleeding from vascular access sites.

Each application includes one (1) StatSeal Disc and one (1) product description label.

Instructions for Use

1. Prepare the skin around the access site per facility protocol.
2. Open sterile packaging and remove disc.
3. Place disc with label side up around the catheter. The disc slit should be perpendicular to the catheter and the edges of the slit should approximate one another to the greatest extent possible.
4. Hold **continuous** pressure over disc for 2 minutes. Extend hold times as clinically necessary.
5. Place transparent dressing per protocol. Apply product description label over dressing.
6. Remove disc with next dressing change.

STATSEAL[®]

Ingredients: Hydrophilic polymer and potassium ferrate

WARNINGS: FOR EXTERNAL USE ONLY. Not intended to replace stitches. Do not apply to visibly infected wounds. May be harmful if inhaled or ingested. Avoid aspirating product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Keep out of reach of children.

CAUTION: Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged or opened. Discard contents after use. After opening, contents are no longer sterile.

**The symbols glossary is provided electronically at:
www.biolife.com/symbols**



Biolife, LLC

8163 25th Court East, Sarasota, FL 34243, USA

+1 941.360.1300 • customer.care@biolife.com



0543



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, Netherlands

Patents 6,521,265, 6,187,347, 8,007,833 and 8,961,479

Other U.S. and international patents pending.