
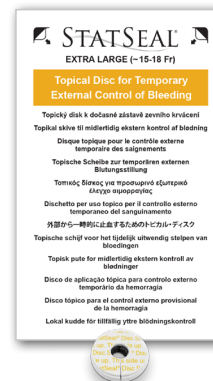
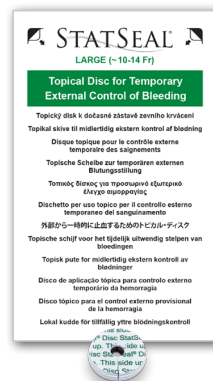
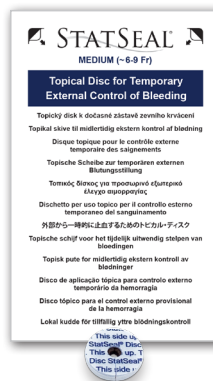


## STATSEAL TOPICAL DISC PRODUCTS

### Disc Application Guidelines

The following disc application guidelines are based on compilation of best practices and can be adapted to your specific application or facility protocol. These guidelines pertain to StatSeal® Small, Medium, Large and Extra Large Disc sizes. StatSeal is intended for use under the care of a healthcare professional for the temporary external control of bleeding wounds and procedures. Each application includes one StatSeal Disc.

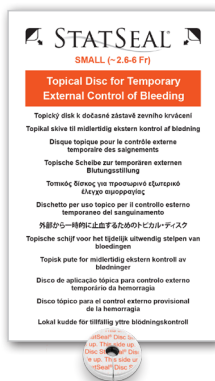

**1**  
Place disc around catheter.

Ensure disc completely surrounds the site.



**2**  
Hold pressure for a **minimum** of 2 minutes.

No peeking! Time it! **Extend pressure times as clinically necessary.**



**3**  
Place transparent dressing.

Leave disc in place until the scheduled dressing change.



### STATSEAL TOPICAL DISC PRODCUTS

#### Disc Removal Guidelines

The following disc removal guidelines are based on compilation of best practices and can be adapted to your specific application or facility protocol.

**1**

#### Remove transparent dressing.

After dressing removal, use dry, sterile gauze to gently brush away any loose StatSeal particles.




Align disc slit with catheter

**2**

#### Clean around site. **Do not disrupt seal.**

Seal should remain when cleaning. Part of seal may come off when cleaning; do not scrub to remove.

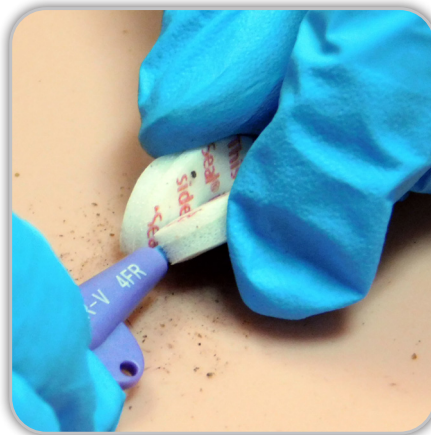


Gently lift disc edges, like 'picking up a taco'

**3**

#### Apply new transparent dressing.

Reapplication of disc is **ONLY** needed if seal has been disrupted and bleeding or oozing occurs.

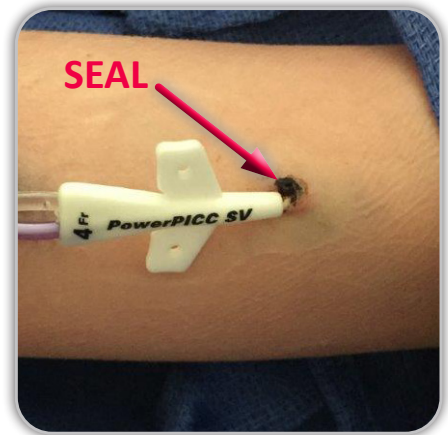

Seal remains at insertion site after cleaning

#### NOTE:

StatSeal has no interaction with CHG. The seal falls off naturally over time with subsequent dressing changes. For catheter removal, moisten dry seal with sterile water/saline to loosen seal.

## BIOLIFE®

Biolife, LLC, maker of StatSeal® products