
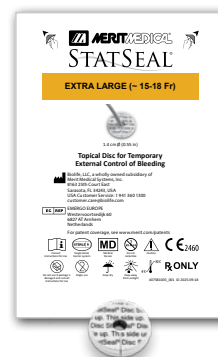
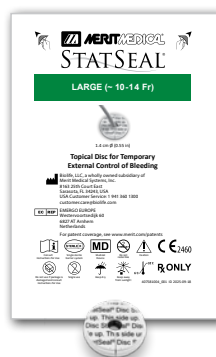
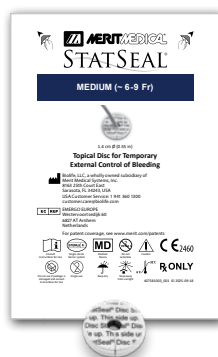


QUICK REFERENCE GUIDE

STATSEAL DISC PRODUCTS

Application Guidelines

The following application guidelines are based on a compilation of best practices and can be adapted to your specific application or facility protocol. These guidelines pertain to StatSeal Small, Medium, Large, and Extra-Large disc sizes. StatSeal is intended for use under the care of a healthcare professional for the temporary external control of bleeding wounds and procedures. Each application includes one StatSeal disc.

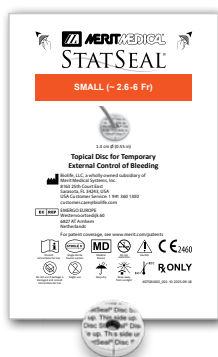

Place the disc around the catheter.

Ensure the disc completely surrounds the site.



Hold pressure for a minimum of 2 minutes.

No peeking! Time it! **Extend pressure times as clinically necessary.**



Place a transparent dressing.

Leave the disc in place until the scheduled dressing change.



Consult Instructions for Use for complete warnings and cautions.

QUICK REFERENCE GUIDE

STATSEAL DISC PRODUCTS

Removal Guidelines

The following removal guidelines are based on a compilation of best practices and can be adapted to your specific application or facility protocol.

1

Remove the transparent dressing.

After dressing removal, use dry, sterile gauze to gently brush away any loose StatSeal particles.



Align the disc slit with the catheter.

2

Clean around the site. **Do not disrupt the seal.**

The seal should remain when cleaning. Part of the seal may come off when cleaning; do not scrub to remove.




Gently lift the disc edges, as if you were "picking up a taco."

3

Apply a new transparent dressing.

Reapplication of a disc is **ONLY** needed if the seal has been disrupted, and bleeding or oozing occurs.

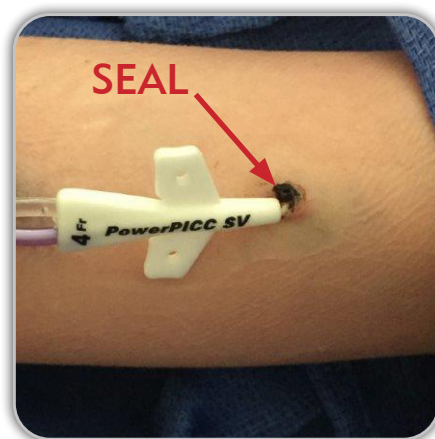

The seal remains at the insertion site after cleaning.

NOTE: StatSeal has no interaction with chlorhexidine gluconate (CHG). The seal falls off naturally over time with subsequent dressing changes. For catheter removal, moisten the dry seal with sterile water/saline to loosen the seal.

The above suggestions and other information, which may include both Merit and third-party products, are for the practitioner's convenience and for general information purposes only. This information does not constitute medical or legal advice, nor is it meant to endorse or guarantee the suitability of any of the referenced products or methods for any specific patient or procedure. Before using any product, refer to the Instructions for Use (IFU) for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.253.1600
1.800.35.MERIT

Merit Medical Europe, Middle
East & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733