


QUICK REFERENCE GUIDE

STATSEAL POWDER & STATSEAL ADVANCED POWDER

Application Guidelines

The following application guidelines are based on a compilation of best practices and can be adapted to your specific application or facility protocol. StatSeal is intended for use under the care of a healthcare professional for the temporary external control of bleeding wounds and procedures.

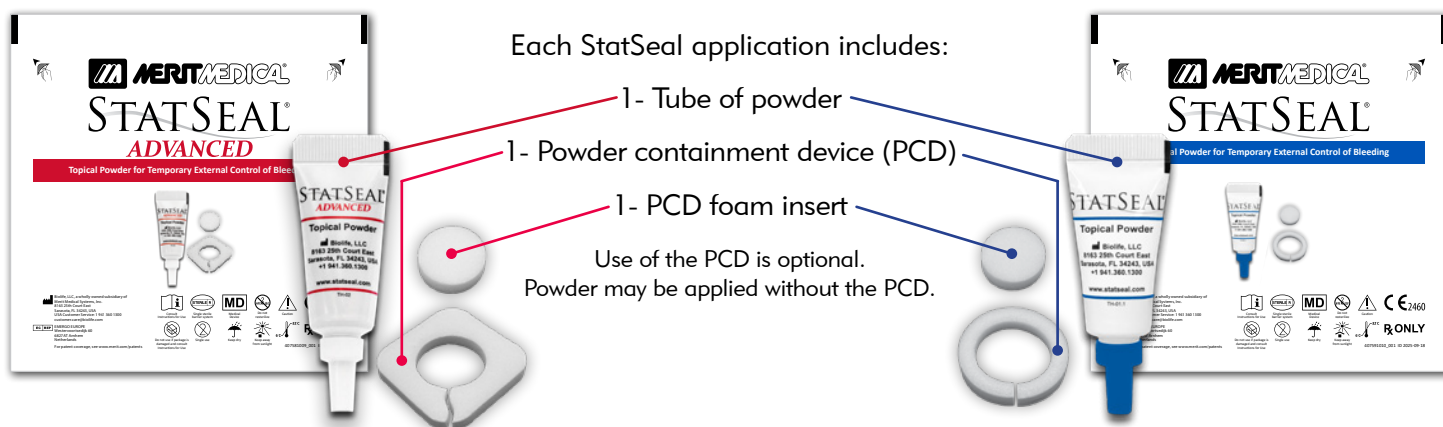

1
Pour powder 360°
around the catheter.

Ensure powder completely
surrounds the site.



2
Hold pressure for a
minimum of 2 minutes.

No peeking! Time it! **Extend
pressure times as clinically necessary.**



3
Place a transparent
dressing.

Leave powder in place until the
scheduled dressing change.



 Consult Instructions for Use for complete warnings and cautions.

QUICK REFERENCE GUIDE

STATSEAL POWDER & STATSEAL ADVANCED POWDER

Removal Guidelines

The following removal guidelines are based on a compilation of best practices and can be adapted to your specific application or facility protocol.

1

Remove the transparent dressing.

After dressing removal, use dry, sterile gauze to gently brush away any loose StatSeal particles.

2


Clean around the site. **Do not disrupt the seal.**

The seal should remain when cleaning. Part of the seal may come off when cleaning; do not scrub to remove.


3

Apply a new transparent dressing.

Reapplication of powder is **ONLY** needed if the seal has been disrupted and bleeding or oozing occurs.

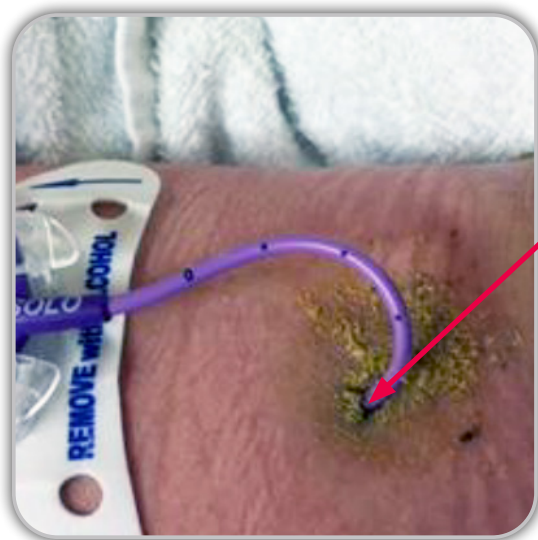

Loose StatSeal remains after dressing removal.

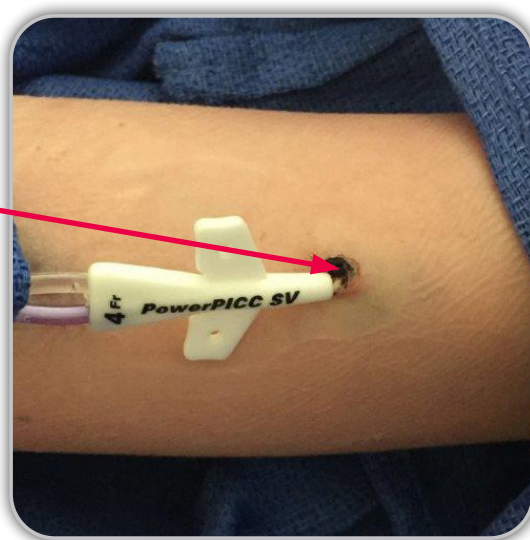

A seal remains at the insertion site after cleaning.

NOTE: StatSeal has no interaction with chlorhexidine gluconate (CHG). The seal falls off naturally over time with subsequent dressing changes. For catheter removal, moisten the dry seal with sterile water/saline to loosen the seal.

The above suggestions and other information, which may include both Merit and third-party products, are for the practitioner's convenience and for general information purposes only. This information does not constitute medical or legal advice, nor is it meant to endorse or guarantee the suitability of any of the referenced products or methods for any specific patient or procedure. Before using any product, refer to the Instructions for Use (IFU) for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.253.1600
1.800.35.MERIT

Merit Medical Europe, Middle
East & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733